


# Important Tips for Optimal CAM<sup>®</sup> Patch Monitoring

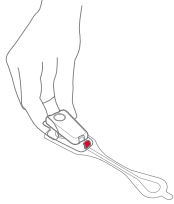

## SKIN PREP

Proper skin prep required to achieve full length of prescribed monitoring duration.

## CONNECTING CAM PATCH

Connect Recorder to CAM patch on a hard, flat surface (such as a table-top) and NOT directly on the chest.

## LOOK FOR LIGHT

Right after connecting Recorder into CAM patch, look for a green light that will blink for 10 seconds.

## PATIENT DIARY

Use your diary to record the date and time of CAM patch activation, as well as each button press associated with your symptoms.